

HeartStart FRx Defibrillator

MAINTENANCE



Green Ready Light



- Blinking green ready light means FRx passed latest self-test and is ready for use
- Solid green ready light means device is in use
- Green ready light off, with or without chirping and flashing blue “i” button, means FRx needs attention

Blue Information Button (“i” button)

- If “i” button is pressed, FRx:
 - is ready for use, or gives troubleshooting information
 - provides CPR coaching
 - provides summary data after an event



Comprehensive Self-Tests



- Regular self-tests
 - Tests done daily, monthly, weekly, and at device power-on
 - Pads integrity test
- Battery insertion self-test
 - Done by inserting a battery
- Verbal update on current status
 - Done by pressing blue “i” button

Battery

- Long Life
 - Five-year shelf life before installation
 - Four-year typical battery life once installed in standby mode
 - 200 shocks or 4 hours of operating time
 - 9 shocks and 15 minutes of operating at first low-battery indication
- Safe and easy to use
 - Just one simple motion to insert into device
 - Send battery back to EHS MFR Services for disposal



Basic Care

- Storage temperature for device (battery installed, pads pre-connected)
 - 32° to 122° F (0° to 50° C)
- Cleaning
 - Soft cloth
 - Approved cleaning agents
 - See Owner's Manual (distributed to all agencies with device)

